



Camera Button Utility Chart

Software	Mode	Time	Key 1	Key 2	Key 3	Notes
Apteryx XVLite, XRayVision, *etc.	Send All Mode	50 ms	SPACE			*Includes other OEM versions of XV and XVLite Only needed for QuickCam Duo and HD
Curve Hero	Send All Mode	50 ms	ENTER			
Dentrix Imaging	Send All Mode	50 ms	ALT+C	ALT+S		
DEXIS	Send All Mode	50 ms	T	SPACE		DEXIS 10 only requires Key 1 to be set.
Eaglesoft	Send All Mode	50 ms	F	S		
EasyDental Imaging	Send All Mode	50 ms	ALT+C	ALT+S		
Kodak Dental Imaging / CS Imaging	Send All Mode	90 ms	F5	F5		
Mogo	Send All Mode	50 ms	F8			
Schick CDR Dicom	Send All Mode	50 ms	ALT+C	U		
SoftDent Imaging	Send All Mode	50 ms	ALT-S			
VixWin Platinum	Send All Mode	50 ms	SPACE	ALT+G	SPACE	
VixWin Pro	Send All Mode	50 ms	ALT+F	ALT+G	ALT+F	
XDR	Send All Mode	50 ms	CTRL+I	CTRL+I		